
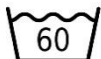







MEYRA GmbH
Meyra-Ring 2
D-32689 Kalletal
Tel. +49 57 33 9 22 - 0
Fax +49 57 33 9 22 - 143
www.meyra.de
info@meyra.de

Cleaning and Care Instructions

Every MEYRA upholstery part and fabric is equipped with a label with washing and care instructions. Please observe those instructions when cleaning your product.

Symbolism

Symbol	Explanation
	Very gentle washing at 30 °C. Fine or wool wash
	Washing at 60 °C
	Do not tumble dry
	Do not iron
	Do not bleach
	No chemical cleaning
	European Conformity

Cleaning and Disinfection

Before cleaning and disinfecting the wheelchair, loose dirt, like food residues or hairs in the area of the castors, has to be removed. Then the detergent solution can be applied. Before disinfecting the detergent must be rinsed off according to the manufacturer's specifications. Then the disinfectant can be sprayed or wiped on and rinsed off if necessary according to the manufacturer's specification.

Detergent Solution

The use of neutral detergents (pH 7) for cleaning is recommended. Acidic (pH 1-6) or alkaline (pH 8-14) detergents can damage plastic components, surface coatings or adhesive joints.

Disinfectants

Disinfectants listed at the Robert Koch Institute are permitted.

The use of neutral disinfectants (pH 7) is recommended. Acidic (pH 1-6) or alkaline (pH 8-14) disinfectants can damage plastic components, surface coatings or adhesive joints.

Cover Parts

Covers for seat and back straps and seat cushions should be washed very gently at 30 °C. Higher temperatures can cause shrinkage of cover fabrics. Foam parts inside the cover should be removed – if possible – before washing due to possible shrinkage during the washing procedure. For foam parts immersion cleaning or immersion disinfection is recommended. Damaged upholstery parts must be replaced. Loop tapes (for hook and loop connections) must be covered in order to avoid surface damage. If it is not possible to cover the loop tapes, hand wash is recommended.

Cleaning and care instructions may vary for individual products. Please observe the instructions in the operating manual of your product.